


Get Started with dB Blockers®

dB Blockers® are custom-made to protect your hearing better than disposable plugs. They may take some time to break in, but with proper care, they will safeguard your hearing for years to come.

HOW TO WEAR

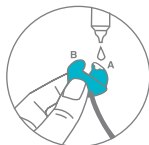

1. Lubrication: Lubricate your dB Blockers™ before insertion the first ten times you wear them with the handy package of lubricant provided.



2. Which one goes in which ear? The right dB Blocker™ is identified by a dB Blocker™ symbol or logo. Hold the dB Blocker™ so the tip (A) faces your right ear.



3. Put in: Gently insert the tip (A) into the ear canal with the helix (B) in a slightly forward position.



4. Twist: Rotate backwards until the helix (B) slips into the pocket at the top of your ear, and is flush with the outside of your ear.

5. Test: Grip the outer part of the ear with your other hand and wiggle it a bit while holding the dB Blocker™ in place. This confirms and fine tunes the blockers into place. Repeat these steps for the left dB Blocker™.

To view a video on this process visit www.protectear.com/howtowear.

EARS CHANGE

Replacement Needed Replace your dB Blockers® every 5 years or if your weight changes by more than 10 lbs. **Important Notice** Do not use non-vented dB Blockers® when flying or scuba diving due to pressure changes. Contact us to order models designed for these activities.

FOR REMOVAL

Break the seal by removing the helix (B) and rotating forward.

⚠ **Do not pull on the cord to remove.**

USE, CARE AND CLEANING

Break-In Period Instructions:

- Lubricate the dB Blockers® before inserting them the first few times.
- Wear them for 2 hours the first day, then add 2 hours daily for a week.
- Use fitting diagrams and alternate hearing protection with old earplugs if needed.
- Clean with warm, soapy water, air dry, and lubricate again.

REPLACEMENTS & ADDITIONAL dB BLOCKERS®

Lost dB Blockers®? Call 1-800-520-0220 with your name and pouch serial number. We have your impressions on file.

WHY CAN I HEAR SO MUCH?

Your dB Blocker® may seem like it's not working, but they are. The vented models allow conversation, alarms, and signals while blocking harmful frequencies.

FIT RIGHT OR WE WILL ADJUST

Your dB Blockers® come with a 90-day FitRight™ Warranty. If there's any discomfort or issues, call 1-800-520-0220.

WARRANTY COVERAGE

Your dB Blockers® are also covered by a 3-year warranty for manufacturing defects (cords and anchors not included).

STAY UPDATED

Get the latest on dB Blockers®, offers, and hearing news.


Phone: 1.800.520.0220 (PST Time Zone)

Email: hear@protectear.com

www.protectear.com

 **Custom Protect Ear®**

WARRANTY

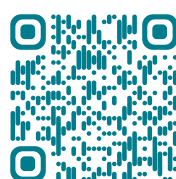

 **dB Blocker®**